EN User manual for Amorphous hydrogel

Description

Amorphous hydrogel in a plastic tube. It needs a secondary dressing to keep it in place and prevent it from drying out. The amorphous hydrogel can both absorb excess exudate from exuding wounds and hydrate dry necrotic tissue or slough.

Indications

Hydrogel may be used in the treatment of chronic and acute wounds including superficial pressure sores, superficial burns, donor sites, recipient skin grafts and dermabrasions. The gel is cool and soothing and can reduce local pain and discomfort in some types of injury.

Contraindications

- Hydrogel should not be used on infected wounds nor on third degree burns. It should also not be used on highly exuding wounds nor on exposed muscle, bones or tendons.
- Do not use if the package is compromised.
- For single use only.
- The product shall be used within the stipulated shelf life.

Method of use

Amorphous hydrogel:
The amorphous hydrogel should be applied directly into the wound up to the level of the surrounding skin and covered with a moisture retentive dressing

Frequency of change

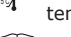
The interval between changes will normally be determined by the degree of exudate produced. Leakage through the second dressing is an indication the gel should be changed. The gel can be rinsed out with sterile saline.

The longest cumulative period of use should not exceed 30 days.

Storage

Hydrogel is stored in a well-ventilated, clean room with a relative humidity of less than 80% temperature 5°C to 35°C and no corrosive gases.

Other

-  Hydrogel is sterilized using gamma radiation.
-  Single sterile barrier system.
-  Do not reuse product.
-  Do not use if package has been compromised.
-  Protect against direct sunlight.
-  Keep dry.
-  Store at a normal room temperature 5 - 35°C.
-  Instructions for use.

DECIFERA® is a registered trademark of Van Heek Medical. All rights reserved.

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